

Novazym http://www.novazym.sklep.pl info@novazym.pl

Dane aktualne na dzień: 02-05-2025 10:42

Link do produktu: http://www.novazym.sklep.pl/vetpcr-iltv-96d-p-637.html



# VetPCR? ILTV (96D)

Numer katalogowy

VET-A009-96D

# Opis produktu

#### Gallid herpesvirus 1 (Infectious Laryngotracheitis virus) Infectious laryngotracheitis virus (ILTV) causes a serious, worldwide-occurring respiratory disease of

chickens which affects growth and egg production and may lead to death of the animals. The acute phase of infection lasts between 1and 2 weeks and is often associated with clinical signs like gasping, coughing, expectoration of bloody mucus, and conjunctivitis. Subsequently, an asymptomatic latent infection of the central nervous system can be established.

VetPCR? ILTV Detection Kit is the direct detection of Infectious laryngotracheitis virus on the basis of a genetic database, so it can diagnose very fast and accurately. It can amplify only specific gene using the PCR (Polymerase Chain Reaction) method, and take only 3 hours for detection. Therefore, it is a very fast, accurate, reliable technique.

#### **Characteristics**

Ready to use : only DNA template and D.W. are needed. Easy and speed protocol. Stable for 1 year at -20?C. Time-saving and cost-effective.

#### Contents

| KIT                                       | Quantity<br>(48) | Quantity<br>(96) | Package |
|---|------------------|------------------|---------|
| VetPCR™ ILTV Premixture                   | 1                | 1                | Vial    |
| PCR Internal Control                      | 1                | 1                | Vial    |
| DNase/RNase free water                    | 1                | 1                | Vial    |
| ILTV PCR Positive control                 | 1                | 1                | Vial    |
| PCR Negative control                      | 1                | 1                | Vial    |
| Mineral Oil Solution                      | 1                | 1                | Vial    |
| Brig <sup>™</sup> Molecular Weight marker | 1                | 1                | Vial    |
| DNA purification kit                      | 50               | 100              | Test    |

## Interpretation of the Test Result

Expected PCR product size : 386bp



Novazym http://www.novazym.sklep.pl info@novazym.pl



### Fig. 1 Result:

Lane M: Brig? Molecular Weight Marker Lane 1~2: ILTV Positive samples Lane I.C.: Internal control Lane P: Positive control Lane N: Negative control