


Link do produktu: <http://www.novazym.sklep.pl/avian-influenza-virus-antigen-test-aiv-ag-1-test-p-429.html>

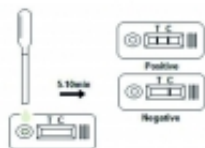

Avian Influenza Virus Antigen Test (AIV Ag), - 1 test

Dostępność

Dostępność - 3 dni

Numer katalogowy

N-81033

Opis produktu

Avian Influenza Virus Antigen Test

Avian Influenza Virus Antigen Test is a sandwich lateral flow immunochromatographic assay for the qualitative detection of Avian Influenza Virus antigen (AIV Ag) in avian's secretions.

Assay Time: 5 - 10 min

Sample: Secretions



Test procedure:

1. Insert the swab stick into bird's cloacae or dip the swab stick in bird's eyes to collect the secretions. Please make the swab wet sufficiently.
2. Insert the wet swab into the provided assay buffer tube. Agitate it to assure good sample extraction.
3. Take out the cassette from the foil pouch and place it horizontally.
4. Gradually drip 3 drops of sample extraction into the sample hole.
5. Interpret the result in 5-10 minutes. Result after 10 minutes is considered as invalid.