


Link do produktu: <http://www.novazym.sklep.pl/poultry-salmonellosis-test-1-test-p-428.html>

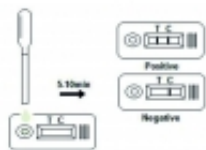

Poultry Salmonellosis Test - 1 test

Dostępność

Dostępność - 3 dni

Numer katalogowy

N-81130

Opis produktu

Poultry Salmonellosis Antigen Test

Poultry Salmonellosis Test is a sandwich lateral flow immunochromatographic assay for the qualitative detection of Salmonella antigen in poultry's serum.

Assay Time: 5 - 10 min

Sample: Serum



Test procedure:

1. Do centrifugation to poultry's blood sample to collect serum for use. If serum is too sticky to move, please do a dilution of 1:1 with PBS or distilled water.

2. Take out the cassette from the foil pouch and place it horizontally.
3. Gradually drip 3 drops of serum/plasma into the sample hole ?S?.
4. Interpret the result in 5-10 minutes. Result after 10 minutes is considered as invalid.